

# Remote Cherry



8 vibration modes  
1 vibration intensity

- \* You can take out the bullet & use it alone
- \* Vibration function can be directly controlled by the button on the bullet.
- \* The operation of the button on the bullet is same as the remote.
- \* But you still need to turn on the remote reception function first, even you don't use the remote.

**Remote** — Put 'Vibration K-Balls' into your body, remote control the vibration functions.

- \* Remote control distance: 10 meter.
- \* Remote use CR1220 button battery.
- \* Remote no waterproof



- \* Please turn on the remote reception function of the bullet before you use the remote.  
Turn on - Press the bullet button for 1.5 seconds till you feel one time short vibration.  
Turn off - Press the bullet button for 2 seconds till you feel two times short vibration.

- \* For saving the bullet's battery, please turn off the remote reception function of the bullet when you don't use it.

- Press the button to turn on it with the 1st vibration mode.
- After turning on, each press the 'Vibration K-Ball' will change a vibration mode, after the 8th mode it will go back to the 1st mode.
- After turning on, anytime quickly press twice the 'Vibration K-Ball' will get into the auto cycle of 8 vibration modes, if want to get out the auto cycle of the 8 vibration modes, just press once.
- Press the button for 1 second to turn off the 'Vibration K-Ball'.



CE RoHS





Length: 119mm Diameter: 36mm

## CE, RoHS certificate

**RoHS and CE Marking Certificate Test Report**

Number: S2N4K003002

Client: ML CREATION CO LIMITED  
 UNIT TEST, BA CHONG CENTRE, NO.50  
 WANG CHANG ROAD, NEW CHANG NEW  
 TERRITORIES, HONG KONG S.A.R.

APPLY: MS SUSANNA LEE

Sample Description: Female Vibrating O-egg

Tests conducted: An examination by the applicant under standard paragraphs details

Concluded: Passed

Remarks: No test was conducted on untested components (T1&T2). All results of these components issued in the report, were referenced to the report S2N4K003002-001 based on No. 20, 22&24

Approved by: Jimmy WEN, Director Manager

**Verification of Conformity**

Test report(s), sample(s) of the better product have been tested to comply with the following based on this verification of the time the tests were carried out. Other tests may be required for the product.

Each direction are verified in compliance, the manufacturer may indicate directions of conformity statements and applying the mark to product intended to be CE compliant with all relevant CE mark Directives requirements.

Applicant: ML CREATION CO LIMITED  
 ROOM 501, BA CHONG CENTRE, NO. 50 WANG CHANG ROAD, NEW CHANG NEW TERRITORIES, HONG KONG S.A.R.

Product: FEMALE VIBRATING O-EGG  
 DC 3V 2V 1.5 V 3V Rechargeable battery  
 with an integral USB (PC) port

Marked: EN 60601-1:2008 + A1:2008 + A2:2011  
 Electromagnetic Compatibility - Requirements for household appliances, health care and similar apparatus - Part 1: Emission  
 EN 60601-1-2:2011 + A1:2011 + A2:2011  
 Electromagnetic Compatibility - Requirements for household product machinery, health and similar apparatus - Part 1: Immunity  
 EN 60601-1-2:2011 + A1:2011 + A2:2011  
 Electromagnetic Compatibility Directive (2004/108/EC)  
 Based on Technical File Entry  
 28 March 2018 to 3 April 2018  
 S2N4K003002-001

Digitally signed by Jimmy WEN, Location: Intertek, Testing Services

**EMC VERIFICATION SUMMARY**

Report No: S2N4K003002-003

Applicant: ML CREATION CO LIMITED  
 ROOM 501, BA CHONG CENTRE, NO. 50  
 WANG CHANG ROAD, NEW CHANG NEW  
 TERRITORIES, HONG KONG S.A.R. HONG KONG

Test Conducted Date: 28 March 2018 to 3 April 2018

ALL TESTS WERE CONDUCTED IN ACCORDANCE WITH  
 EN 60601-1:2008 + A1:2008 + A2:2011  
 EN 60601-1-2:2011 + A1:2011 + A2:2011

| Test           | OK                                  | Not OK                   | Remark |
|----------------|-------------------------------------|--------------------------|--------|
| 2009 + A2:2011 | <input checked="" type="checkbox"/> | <input type="checkbox"/> |        |
| 2011 + A2:2008 | <input checked="" type="checkbox"/> | <input type="checkbox"/> |        |

Conclusion: The Measurement Uncertainty of test has been considered

Approved By: Jimmy WEN, Engineer, Intertek, Testing Services, Date: 2 April 2018

### Modello di giocattolo del sesso di creazione ML Remote Cherry

**Forte vibrazione - Telecomando - Uova da donna**  
 Per coppie o donne

Il vibratore può essere rimosso per il massaggio del clitoride

Un uomo può usare un telecomando per controllare la modalità di vibrazione dell'uovo all'interno di una donna

100% impermeabile e può essere pulito normalmente

Uovo vibratore, uovo vibratore remoto, miglior uovo vibratore con telecomando

Fabbrica di giocattoli sessuali Cina, fabbrica di giocattoli sessuali, produttore di giocattoli sessuali, fornitore di giocattoli sessuali